HydCAL

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Uso exclusivo del dentista

HydCAL

Radiopaque Calcium Hydroxide Cement

Indication: It is indicated for pulp capping and as a protective liner under restorative materials or other cements.

Technical specifications: Hydcal is a self-setting and rigid composition made of calcium hydroxide, by the reaction of a base paste and a catalyst paste, which should be mixed at the moment of use. Hydcal does not inhibit the polymerization of composite resins used for restorations. Also, it has high resistance to dissolution by the phosphoric acid used for acid etching of dental structures. Hydcal is produced in a color similar to the dentin in order to avoid any influence to the color of restorative materials. Hydcal is radiopaque

Composition: Base Paste: Ester Glicol Salicilate, Methyl Salicilate, Barium Sulfate, Titanium Oxide, Silicon oxide and Iron Oxide Colorant. Catalyst paste: Calcium Hydroxide, Zinc Oxide, Iron Oxide Colorant, Zinc Estearate, Etil-toluene Sulphonamide and Purified Water

Validity: 3 years from manufacture date

No contraindications

Instructions for use

- In a mixing pad, use equal lengths of base paste and catalyst paste
- Then, using a spatula, mix the two pastes in order to obtain a uniform color.
- To guarantee a higher working time, try to complete the mixture in 10 seconds
- Put the mixed material inside the cavity, that must be dry, finishing the insertion before the material starts to set.
- In room temperature and normal humidity, the material sets within 2 to 3 minutes in the mixing pad. Inside the mouth, the setting time is lower because of mouth's humidity and temperature.
- The excess material in retentive areas or margins may be removed with curettes or explorers, after the material finishes setting.

Warning: The essential alkaline nature of the material allows it only to be used when it can be protected by the oral environment. The material can not be used as temporary restorative material or for cementation. It is important to keep the proportion between base and catalyst in order not to change material's radiopacity, resistance, durability and setting time.

Storage: Keep at room temperature or below. Refrigeration increases product durability. Let the material return to room temperature before using it.

Exclusive for dentist's use