Single - Temporary Filling Material

USE INSTRUCTIONS

Indication:

Newbond is a self-setting filling material, radio-opaque, for the temporary filling of tooth cavities. Newbond is a Zinc Oxide-based cement designed for short period temporary applications, to be used for no longer than 2 weeks, not only for the temporary filling of Class I and II cavities but also for the temporary sealing in Endodontics.

Product technical characteristics and specification:

Composition: Zinc oxide, Zinc sulfate, Cálcium sulfate hemi-hydrated, Dibutyl adipate, Polyvinyl chloride copolymer, Iron oxide dye and Mint aroma.

Hardening:

Newbond's surface hardens within 20 to 30 minutes by water absorption (for example: by the action of saliva). After 2 or 3 hours, the temporary filling may be submitted to masticatory force.

Procedure:

Remove the amount of Newbond appropriate for the size of the cavity with a spatula and put it on a mixing plate. Observation: Immediately close the pot, in order to avoid the surface to harden. Moisten the cavity with water spray, apply, Newbond to the cavity with a condensing instrument and adapt it to the cavity's walls with a smooth condensation.

Remove occlusal contact points with a scaler.

Newbond can be removed from the cavity by using burs or scalers. The cement residuals can be washed out of the cavity with water.

Special cautions:

- Newbond should be stored hermetically closed in a temperature not higher than 23° C and with 50% of humidity.

- Avoid direct sun exposure and other heat sources.

- Only dentists should handle Newbond, and they will determine the treatment duration according to individual needs.

- In case of eye contact, wash carefully with water for approximately 10 minutes and contact an ophthalmologist.

- Keep out of the reach of the children.

- Close the pot hermetically after each application.

Valid date: 2 years

ANVISA register: 80015520012

NewBond®

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Uso professional
Uso Profesional
Professional Use

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ATTENTION:

*Material for filling
of dental cavities*

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